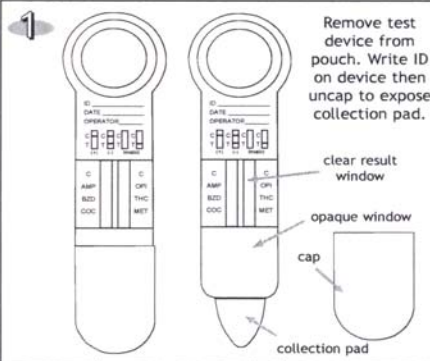
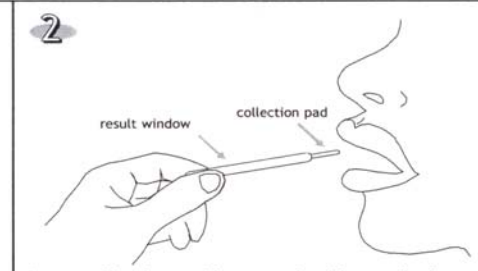
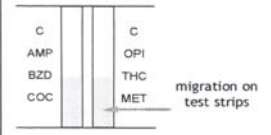
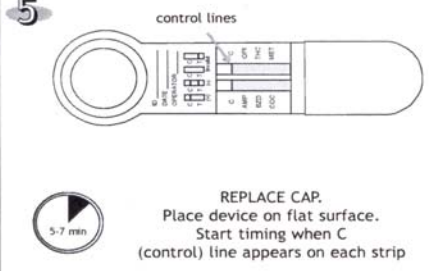
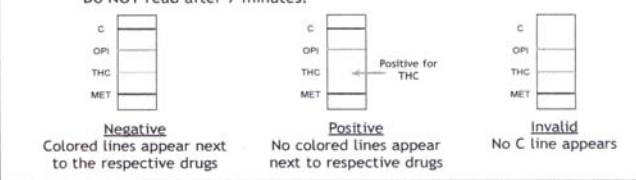
SCHEDA RAPIDA SALIVA TEST MULTIDROGA ORAL-VIEW™

ORAL-VIEW® Saliva Multi-Drug of Abuse Test PROCEDURE CARD

<p>1</p>  <p>Remove test device from pouch. Write ID on device then uncap to expose collection pad.</p> <p>clear result window</p> <p>opaque window</p> <p>cap</p> <p>collection pad</p>	<p>2</p>  <p>result window</p> <p>collection pad</p> <p>Insert collection pad into mouth with test device inserted under the tongue and with lips of mouth touching opaque window but not touching clear results window.</p>	<p>3</p>  <p>Do not bite, chew or pull on collection pad.</p>
<p>4</p> <p>Leave in mouth under the tongue. After 2 minutes, remove the device from the mouth. If you do not see a pink migration in the result window, put the device back in the mouth under the tongue for 1 additional minute then remove.</p>  <p>C OPI AMP BZD THC MET COC migration on test strips</p>	<p>5</p>  <p>control lines</p> <p>REPLACE CAP. Place device on flat surface. Start timing when C (control) line appears on each strip</p> <p>5-7 min</p>	<p>6</p> <p>Read results at 5-7 minutes after the C line appears on EACH strip. Do not compare one line intensity to another. Do NOT read after 7 minutes.</p>  <p>Negative Colored lines appear next to the respective drugs</p> <p>Positive No colored lines appear next to respective drugs</p> <p>Invalid No C line appears</p>

13200 Gregg Street, Poway, CA 92064
PH: 858-513-3888 • FAX: 858-513-8388
www.alfascientific.com • info@alfascientific.com
(English) 31-4262C

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